Simeon
Ranger
Listen Further

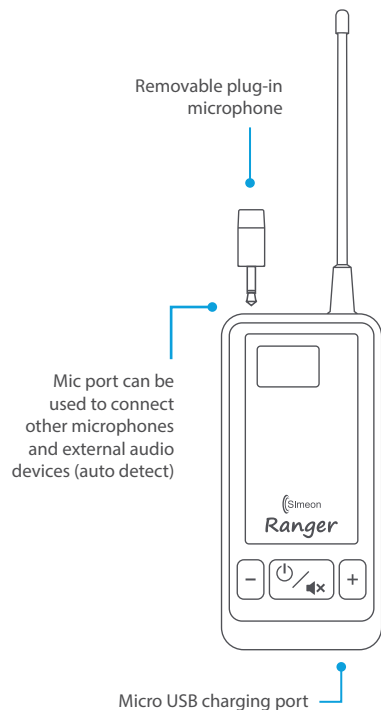
User Manual – Page 1
Manuel de l'utilisateur – Page 5
Manual de Usuario – Pagina 9

Thank you for choosing the Simeon Ranger remote microphone system. Your Ranger system uses UHF wireless radio technology to help you hear better across a distance and in noisy environments. Ranger has 96 transmission channels, extended operating

range, higher volume and studio-quality sound. It can be used with many types of listening accessories and can be coupled wirelessly with most hearing aids and cochlear implants.

Included with your Ranger system:

- Ranger transmitter with mini microphone
- Ranger receiver
- Device holster with lanyard (2)
- Dual micro USB charger
- Earphones
- Carry case
- User manual

Ranger Transmitter

When listening in noisy environments, have the speaker wear the transmitter with the plug-in microphone within 25cm of their mouth. For even better performance in noise, use an optional boom (headset) microphone with the transmitter. This will capture the speaker's voice at just a few centimeters from the mouth.

External audio devices such as computers and phones can be connected directly to the microphone input port with a cable. Your Ranger transmitter will detect this audio signal and automatically adjust the signal level.

You can make fine volume adjustment with the transmitter using the -/+ keys. The full range of volume adjustment is made on the receiver.

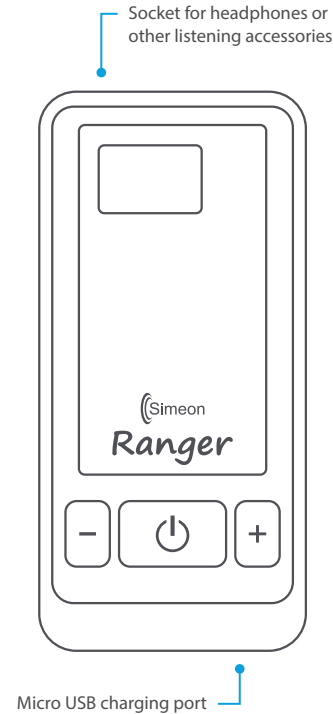
Ranger Receiver

You can listen to your Ranger receiver with stereo headphones, earbuds and a range of specialized listening accessories.

⚠ Caution - avoid excessive volume levels. Listeners with normal hearing sensitivity should use low-output (attenuated) earphones for prolonged use of Ranger. The receiver volume control can be locked with the keypad to prevent unintended volume changes.

If you have telecoil-equipped hearing aids or cochlear implants, use the neckloop or other optional Ranger coupling accessories to connect to your device wirelessly. You may have to select a 'T' or 'MT' setting on our personal equipment. Consult with your hearing professional to select the best listening accessory for your personal equipment.

Note that when no transmission is detected for 20 minutes, the receiver will turn off automatically to conserve the battery power.

Using the Ranger Keypad

Power On/Off	Press and hold  for 2 seconds
Volume Control	Increase  Decrease 
Volume Control Lock/Unlock	Press and hold   together for 2 seconds
Microphone Mute/Unmute	Press  briefly
Change Channel	Press and hold   together for 2 seconds
	Select a channel using  or 
	Press  briefly to confirm the channel selection

Charging your Ranger

Use the micro USB cable to charge the system. A full charge from empty may take six hours. The battery icon will stop flashing when charging is complete. Before storing your Ranger for prolonged periods, fully charge the transmitter and the receiver. Special dual-pocket and multi-pocket charging stands are available for multiple Ranger units.

Warranty

Your Simeon Ranger has a 3 year warranty as standard, effective from the date of purchase. Peripherals including microphones, chargers, adapters and batteries are warranted for one year.

The manufacturer's warranty provides for repair or replacement of defective components or workmanship. Expressly not covered are damages resulting from water damage and the unapproved substitution of components, batteries or accessories. Extended warranty coverage may be purchased under the Simeon Service Plus extended full service program.

Notice: This device complies with Industry Canada license-exempt RSS standard(s). This device complies with Part 15 of the FCC rules. Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.

The changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Important note: To comply with the FCC RF exposure compliance requirements, no change to the antenna or the device is permitted. Any change to the antenna or the device could result in the device exceeding the RF exposure requirements and void user's authority to operate the device.

Technical Specifications

Overall System	
Frequency Preparation	PLL synthesized control OSC, 96 UHF channels
Modulation	Wideband FM
Carrier Frequency Range	640 ~ 664 MHz (Ranger 1.0); 514 ~ 544 MHz (Ranger 1.1)
Nominal/Peak Deviation	±20 KHz/ ±40 KHz
Frequency Response	40 Hz ~ 18 KHz
Dynamic Range	> 80dB
THD at 1KHz	< 1%
Service Coverage	Up to 100m (open field)
Operating Temperature	-10°C ~ 50°C

	Ranger T1.0 / T1.1 Transmitter	Ranger R1.0 / R1.1 Receiver
RF Output Power	10mW	
Antenna Type	Whip antenna	Headphone antenna
3.5 Phone Jack	Mic/Aux In	Headphone output
Operating Time	12 hours	18 hours
Battery Type	1300mA 3.7V Li-Polymer Rechargeable	
Charging Source	DC 5V Micro USB	
Charge Time	< 6 hours	
LCD Display	Channel, Battery indicator, Lock, Volume	
Operating Temperature	FCC / Industry Canada	
Dimension	82.6(H) x 40(W) x 12(D)mm Antenna length not included	82.6(H) x 40(W) x 12(D)mm
Weight	49 g	

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Supportive Hearing Systems, Inc.