
Simeon | Supportive Hearing Systems Inc.

Audita Getting Started Guide

Audita is available in both FM and Digital configurations. Check which Audita version you have on the device itself.

Every Audita is shipped with batteries fully charged. The transmitters and receiver are already paired and channels pre-assigned. Your Audita is all ready to turn on and use straight out of the box!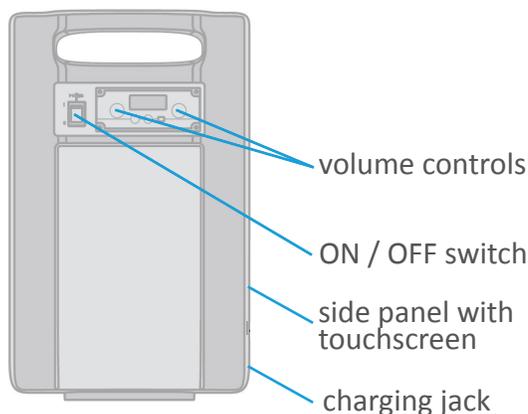

The main controls for Audita are on the front of the wireless receiver / speaker - these are all you need to use your system.

Switch the receiver / speaker ON and turn the two volume control units to a moderate setting. These will control the level of each microphone.

There are special controls and a charging jack on the side panel. External devices can be connected at the side panel too. Settings for external devices and other special settings are accessed through the side panel touchscreen.

audita wireless receiver / speaker

accessory microphones may be plugged into the jack here

built-in microphone works best about 20 cm from your mouth

press and hold for ON / OFF
press quickly to MUTE (red light) and UNMUTE (blue light)

press quickly to MUTE (red light) and UNMUTE (blue / green light) the switch on back of the transmitter / microphone

press and hold for ON / OFF

pendant transmitter / microphone

handheld transmitter / microphone

Simeon | Supportive Hearing Systems Inc.

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