User Manual - Page 1
Manuel de l'utilisateur - Page 24
Manual de Usuario - Pagina 47

Table of Contents

| | |
|---|----|
| Read before use | 2 |
| General information | 3 |
| Introduction | 4 |
| Product components and included accessories | 4 |
| Using your Sprek | 5 |
| Using the boom (headset) mic | 7 |
| Speaker ON / OFF | 8 |
| Transmitter ON / OFF | 9 |
| Battery level indicator | 10 |
| Volume control | 11 |
| Pairing the transmitter(s) with the speaker | 12 |
| Using dual transmitters | 14 |
| Setting a channel number | 15 |
| Streaming audio and music | 16 |
| Charging the transmitter and speaker | 19 |
| Speaker reset function | 20 |
| Transmitter reset function | 21 |
| Technical specifications | 22 |
| Simeon Warranty | 23 |

Before Use

Notes

- Check you have all the system components (see below)
- Charge the transmitter and speaker fully before first use
- Use only the recommended charging accessories
- Avoid excessive sources of heat
- Do not immerse in water

General Information

| | |
|--------------------------------------|---|
| Digital Transmitter | <ul style="list-style-type: none">• Charge fully before first use using the charging adapter and USB cable provided.• The digital transmitter is supplied with a gooseneck and a boom (headset) microphone.• In most circumstances the boom (headset) microphone will offer the best performance. The gooseneck microphone offers convenience and must be positioned within 20cm of the mouth to perform well.• Avoid pointing the microphone directly at the speaker or feedback (squealing sound) may occur when you turn the system on. Select a low volume level initially and slowly increase the volume to a comfortable listening level.• If you chose to use the headset microphone plugged directly into the speaker (see below for 'hard-wired' operation) make sure the digital transmitter is turned OFF. |
| Boom Mic (Headset Microphone) | <ul style="list-style-type: none">• The boom microphone picks up speech close to the mouth, improving performance in noisy environments, and will help to avoid any acoustic feedback.• Place the mic frame earhooks over your ears and adjust the boom arm to position the mic capsule to point to the corner of your mouth.• It is not necessary to place the microphone in front of your lips and doing so may diminish sound quality.• The boom microphone frame can be adjusted to rest comfortably over both ears, with no other contact or pressure points against the head or face.• Plug the microphone into the jack on top of the digital transmitter• Alternatively, the boom microphone may be plugged directly into the AUX IN port of the speaker for 'hard-wired' operation. In this mode of operation the digital transmitter should be turned OFF. |
| Gooseneck Microphone | <ul style="list-style-type: none">• The gooseneck microphone may be used if the digital transmitter is worn around the neck on the lanyard supplied with the system.• Position the microphone so the capsule is within 20cm from your mouth. Adjust the flexible gooseneck away from your body so that the microphone windscreen does not rub against your clothing.• Plug the microphone into the jack on top of the digital transmitter.• It is not recommended to plug the gooseneck microphone directly into the speaker. |
| Speaker Connections | <ul style="list-style-type: none">• Use the charging adapter and USB cable provided with the Sprek speaker.• The speaker has an AUX IN port which will accept most Simeon microphones as well as peripheral audio devices.• The AUX IN port does not support consumer smartphone earphones or headsets. |
| Bluetooth | <ul style="list-style-type: none">• Note that the digital transmitter communicates with the speaker via a 900MHz real-time wireless signal. The Bluetooth function is used only for streaming audio from smartphones and other devices.• When pairing your smartphone or other Bluetooth device with the speaker to stream music etc., keep them less than 10m apart. |

Introduction

Sprek is an ultra-compact toteable soundfield system that can be used for special hearing assistance needs as well as for personal voice amplification.

Operating on 900MHz license-exempt digital wireless band (North America only), Sprek is not subject to 2.4GHz band wireless interference.

The full-range speaker offers excellent speech intelligibility and 15 Watts of output power.

Product Components


Wireless speaker

Transmitter


Boom microphone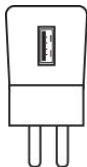


Charging adapter

USB Y-cable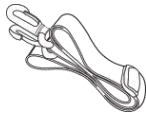


Shoulder Strap

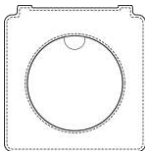
Speaker pouch


Using Your Sprek

Sprek speaker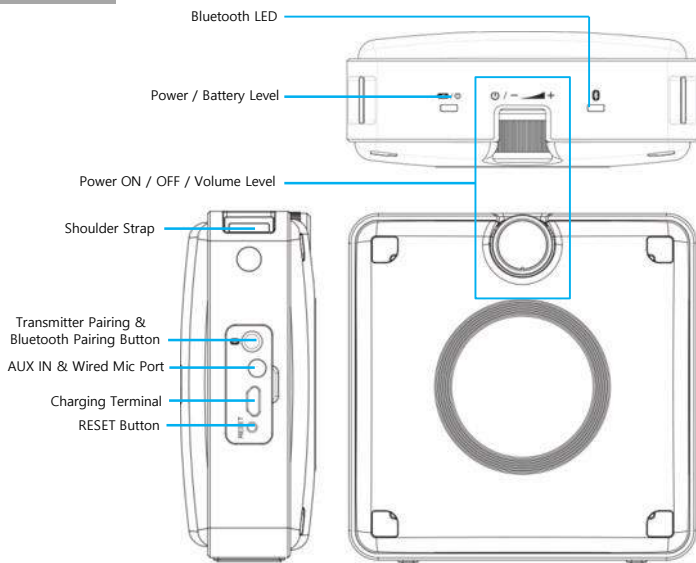



How to Use Shoulder Strap

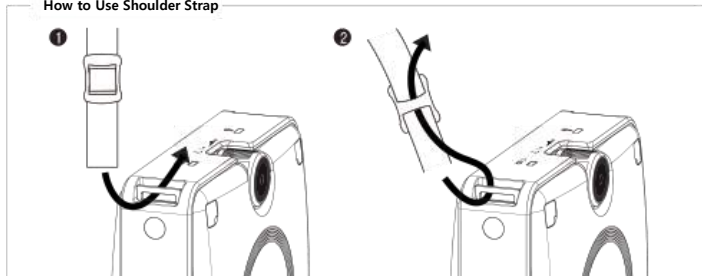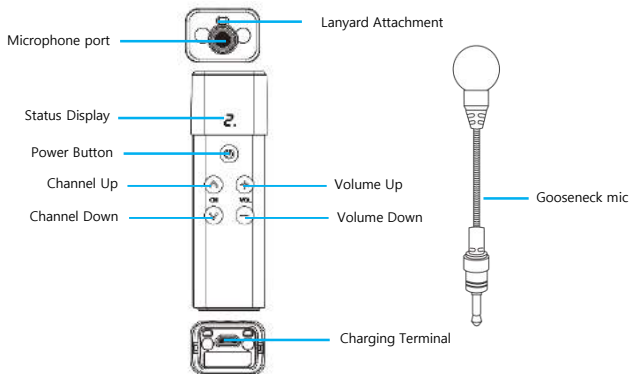
How to connect a microphone

- 1 Insert the gooseneck microphone into the mic port on top of the transmitter
- 2 Insert the boom (headset) microphone into the mic port on top of the transmitter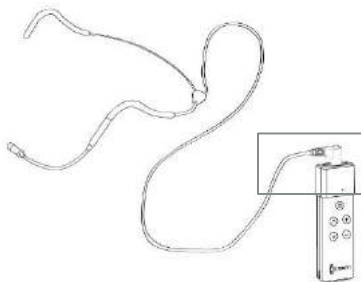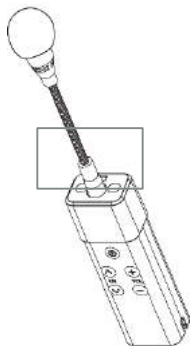


Using the Boom (Headset) Microphone



A unidirectional boom (headset) microphone is supplied with your Sprek system as a premium accessory. The boom microphone picks up speech very closely, improving performance in noisy environments, and will help to avoid any acoustic feedback. In most circumstances, the boom microphone will provide better performance and more available amplification than the standard gooseneck microphone. It is important to position the boom microphone correctly and adjust it for user comfort..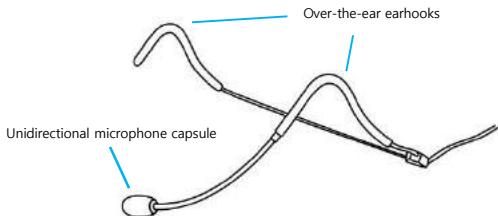


- 1 Connect the boom microphone into the mic port on top of the transmitter. Alternatively, the boom microphone may be plugged directly into the AUX IN port on the side of the Sprek speaker for 'hard-wired' operation
- 2 Adjust the headset frame so the earhooks rest comfortably over both ears, with no other contact or pressure points against the head or face.
- 3 Adjust the boom arm to position the mic capsule to point directly towards the corner of your mouth. The best angle of pickup is within a 30° range. It is not necessary to place the microphone in front of your lips and doing so may diminish sound quality.

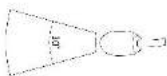Speaker On / Off / Volume

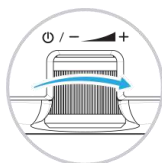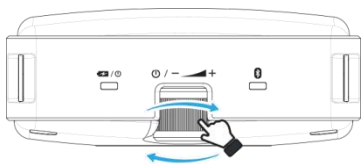Power ON

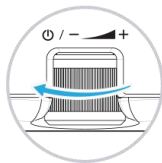
Power OFF

Note: If left unused for 30 minutes, the speaker will automatically power off to conserve power


Speaker LED indicator

LED blinks differently depending on transmitter connection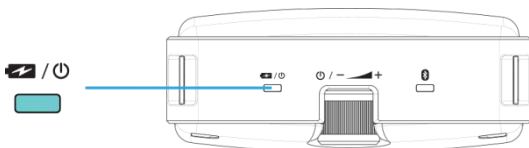


When transmitter is connected, LED blinks twice quickly

When transmitter is not connected, LED blinks slowly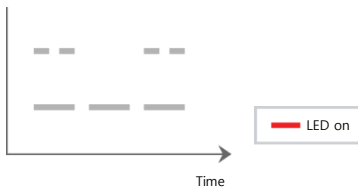


Transmitter On / Off

Press and hold the power button for **two seconds** to turn ON or OFF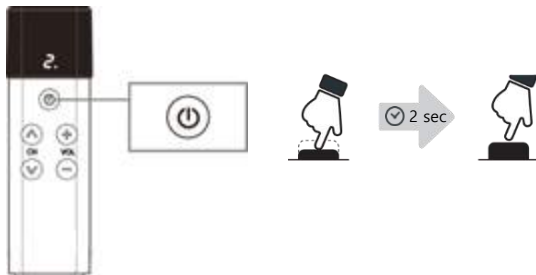

Battery Level Indicator


The blinking red battery LED shows the remaining battery level for the speaker. The number of LED blinks (one, two or three) indicates the approximate remaining battery charge.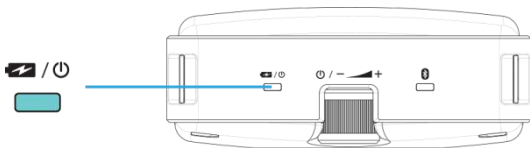



Three LED blinks = $\geq 80\%$ battery

Two LED blinks = $\sim 50\%$ battery


One LED blink = $\leq 20\%$ battery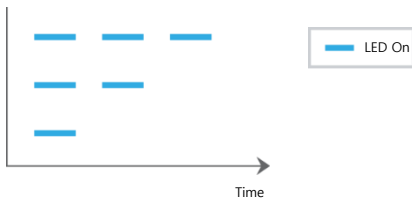


The transmitter has three LED bars to indicate the approximate remaining battery charge.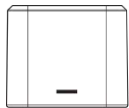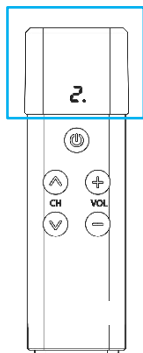

One bar = $\leq 20\%$ battery

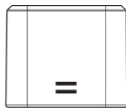
Two bars = $\sim 50\%$ battery

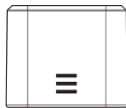
Three bars = $\geq 80\%$ battery

Volume Control

The volume level can be controlled from the speaker or from the transmitter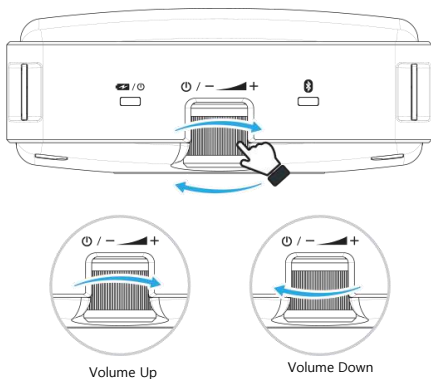


The transmitter has a mute function as well as volume adjustment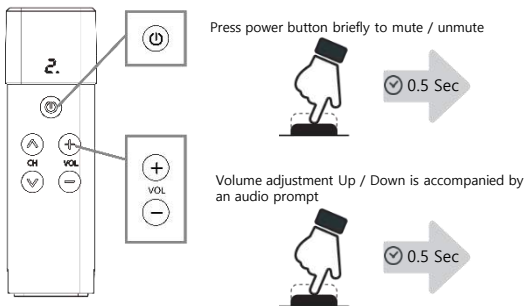

Pairing the Transmitter(s) with the Speaker

To set up for dual-transmitter operation, both transmitters must be in pairing mode before the speaker pairing sequence is begun.


- 1 Enter Transmitter Pairing Mode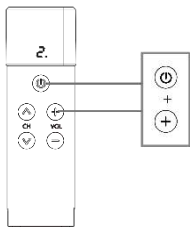


Press together

Note: You can also enter transmitter pairing mode by pressing and holding the power button continuously for 10 seconds.

First transmitter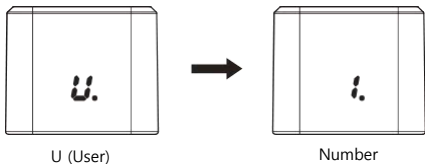


Second transmitter

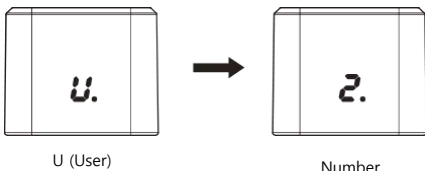
2 Enter Speaker Pairing Mode

To prepare the Sprek speaker for pairing with one or two transmitters, you must start the speaker pairing sequence. First, press and hold the pairing button **before turning the speaker ON**. Keep the pairing button pressed while turning the speaker ON - an audio prompt "Pairing" from the speaker will confirm that you have started the pairing sequence.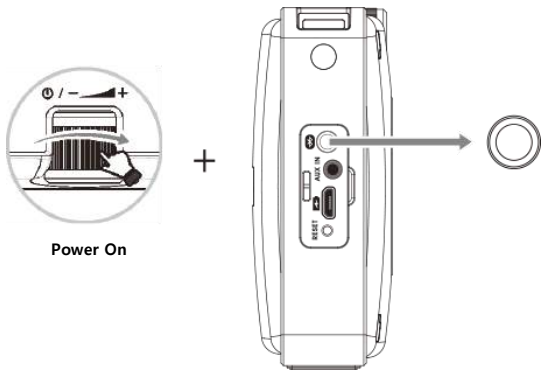

3 To indicate that pairing is complete, the red LED will fade out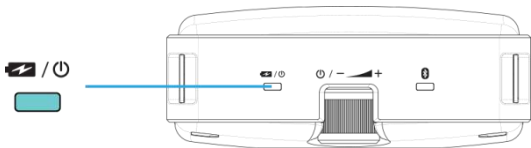


Note: You can pair a number additional Sprek speakers with your system to expand coverage for larger spaces

Using Dual Transmitters

There is no need to assign a transmitter user number for normal single-transmitter operation. If you add a second transmitter to your Sprek system, it will be necessary to assign a separate number (user '1' and user '2') to permit dual operation without interference.


Setting user number for dual-transmitter operation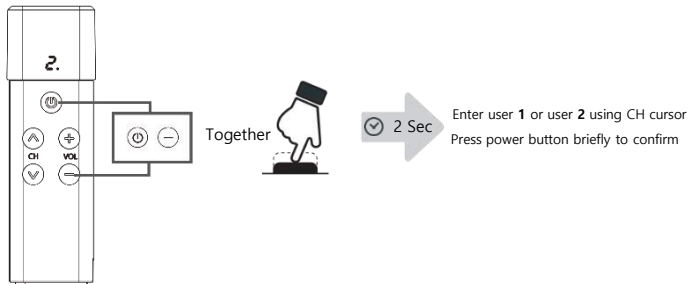

▲ Important

- For dual-transmitter operation, the two transmitters must be assigned user number **1** and user number **2** to avoid signal interference.
- To pair two transmitters with the speaker, both transmitters must be in pairing mode (see below) before the speaker pairing sequence is begun.


User number display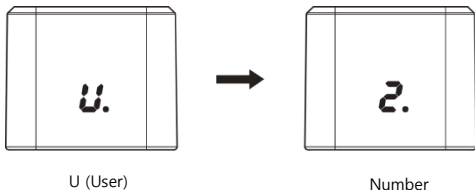

Setting a Channel Number

If you are using multiple Sprek systems in the same building, you should assign a different channel number (between 0 and 9) to each system to avoid frequency interference. Note that when using dual transmitters with a speaker, the transmitters should be set to the **same** channel number but assigned as user '1' and user '2' to avoid interference.


Setting channel number using the transmitter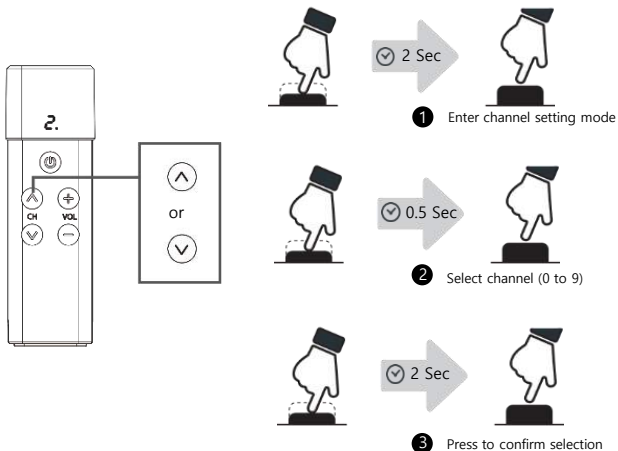


Streaming Audio and Music

You can stream music and other audio from your smartphone or computer wirelessly through the Sprek speaker. First you must pair your device with the Sprek speaker.


1 Pairing with Smart Phone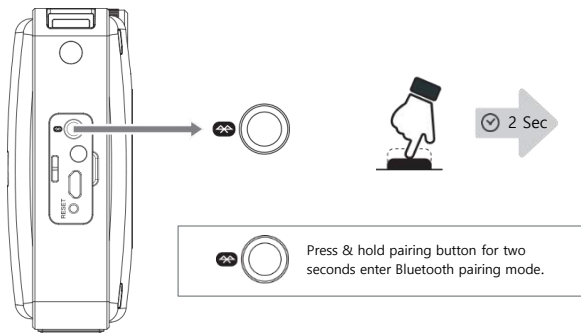


iOS Phone

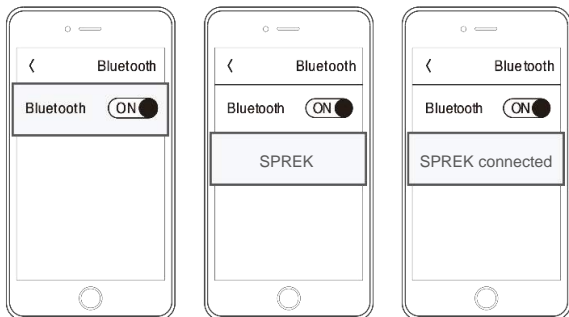With Android phone

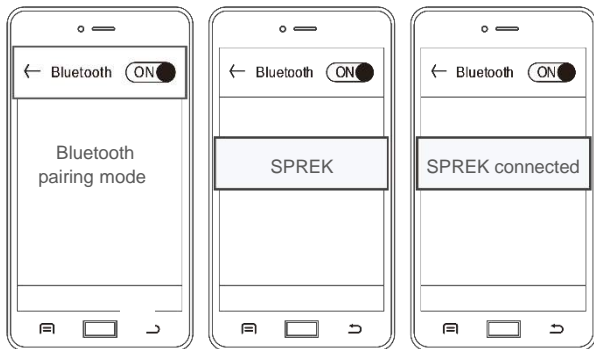
2 Bluetooth Mode

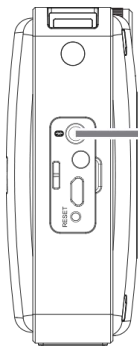
After Sprek is paired with your smartphone, it will be easy to resume the Bluetooth connection in the future by pressing the pairing button briefly.

Streaming music through your Sprek speaker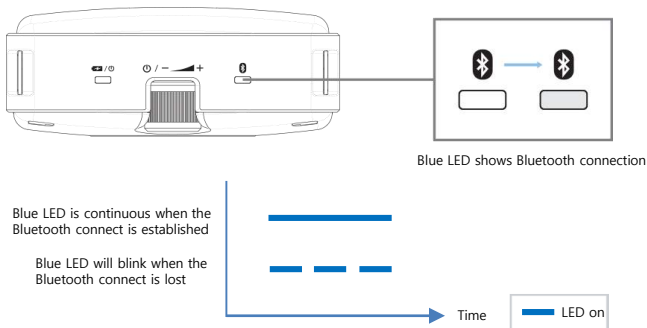


Streaming music can even be controlled from the Sprek transmitter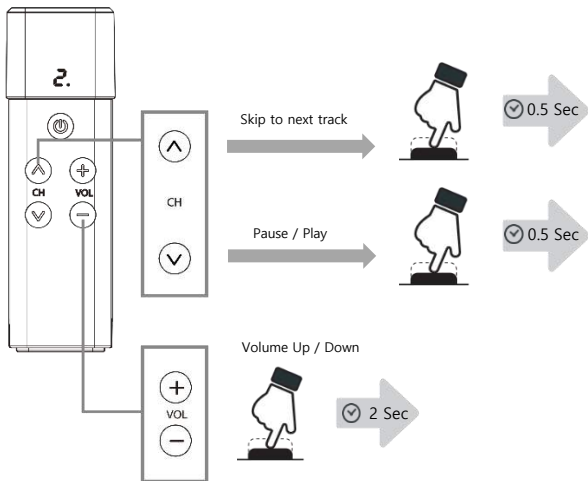

Charging the Transmitter and Speaker Together

Speaker: Battery LED turns orange while being charged and turns green once charging is complete.

Transmitter: Blue and red appear together during charging and only blue once charging is complete.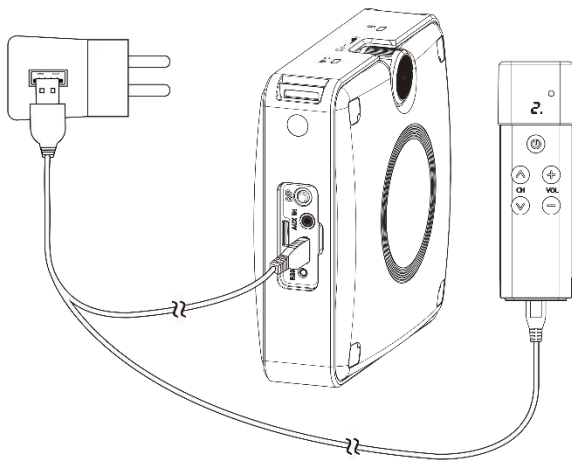


Notes

- Use only the recommended power adapter (output 5V/2A)
- Turn transmitter and speaker OFF during charging
- Avoid prolonged charging to maintain optimum battery condition

Speaker Reset

The speaker RESET button will reboot the speaker and restore settings

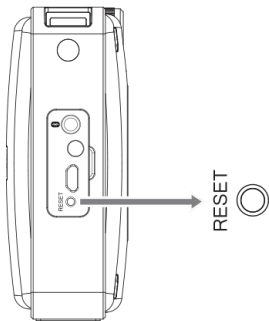
The RESET button is recessed to prevent accidentally activation. Use a sharp object such as a ballpoint pen to access the RESET button

Transmitter Reset

- 1 Press channel down (V) and volume up (+) together to enter transmitter reset mode

- 2 Press and hold the power button for two seconds to confirm transmitter reset

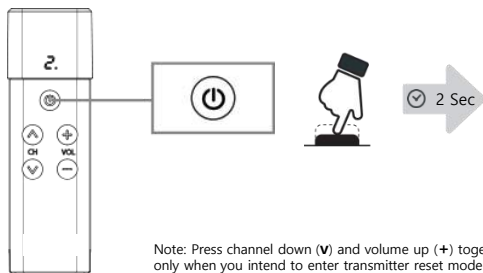Technical Specifications

| | |
|--------------------------|--|
| Transmission frequency | 902MHz to 928MHz |
| Transmission method | TDMA |
| Modulation system | FSK |
| Transmitter battery | Rechargeable Li-ion 3.7V / 650mAh |
| Wireless speaker battery | Rechargeable Li-ion 3.7V /3,400mAh |
| Autonomy* | Speaker 15 hours; Transmitter 9 hours |
| Charging Time | Speaker 6 hours; Transmitter 2 hours |
| Operating temperature | -10 to +50°C |
| Dimensions | Speaker: 105 x 105 x 40mm Transmitter: 28 x 93 x 20mm |
| Weight | Speaker 340g; Transmitter 60g |
| Audio frequency response | 100Hz ~ 20kHz |
| Speaker output | 10W continuous; 15W max. |
| Bluetooth media streamer | Bluetooth version 4.1 |
| Bluetooth Function | A2DP, AVRCP |
| Certifications | Industry Canada, FCC |

* Speaker autonomy measured at volume level 2

Simeon Sprek Warranty

Your Simeon Sprek system has a 3 year warranty as standard, effective from the date of purchase. Peripherals including microphones, chargers, adapters and batteries are warranted for one year.

The manufacturer's warranty provides for repair or replacement of defective components or workmanship. Expressly not covered are damages resulting from water damage and the unapproved substitution of components, batteries or accessories. Extended warranty coverage may be purchased under the Simeon Service Plus extended full service program.

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Supportive Hearing Systems Inc.